FERRIS

PolyMem®

"The Pink Dressing"™


Polymeric Membrane Dressing
Vendaje de Membrana Polimérica
Polymerer Membranverband
Pansement à Membrane Polymérique
Medicazione a membrana polimerica
Polymermembran bandage
Polymeer schuimverband

Directions for use
Instrucciones de uso
Gebrauchsanleitung
Mode d'emploi
Modalità d'uso
Brugsanvisning
Gebruiksaanwijzing

Latex Free Products and Packaging
Producto y Empaque libres de latex
Latexfreie Produkte und Verpackungen
Produits et emballage exempt de latex
I prodotti e le confezioni non contengono lattice
Producten en verpakking bevatten geen latex
Latexfreie produkter og emballage

STERILE R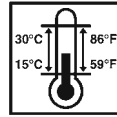

AU REP

Emergo Asia Pacific Pty Ltd
201 Sussex Street
Darling Park, Tower II, Level 20
Sydney, NSW 2000 Australia

EC REP

Emergo Europe
Molenstraat 15
2513 BH The Hague
The Netherlands

Made in U.S.A. by:
Fabricado en los EE.UU. por:
In den USA hergestellt von:
Fabriqué aux USA par:
Fabbricato in U.S.A. da:
Fabrikeret i USA af:
Gemaakt in de U.S.A. door:

FERRIS MFG. CORP. • 16W300 83rd Street • Burr Ridge, IL 60527-5848 U.S.A.
630-887-9797 Toll Free U.S.A.: 800-765-9636 International: +1 630-887-9797
e-mail: info@ferrispolymem.com Website: www.PolyMem.eu Website: www.PolyMem.com

Covered by one or more U.S. and similar international patents: 4,884,563; 5,064,653; 5,065,752; 5,254,301; 5,916,928; 5,605,165; 6,043,406; 6,346,653; 6,349,828; 6,350,339; 7,078,055; 7,078,056. Other patents pending

FERRIS

PolyMem[®]

"The Pink Dressing"[®]

PRODUCT DESCRIPTION

The Ferris PolyMem Wound Dressing is composed of a hydrophilic polyurethane membrane matrix with a semipermeable polyurethane continuous thin film backing which is optimized for oxygen and moisture vapor permeability and as a barrier to liquids. It is the **only** dressing which contains (1) a safe, non-toxic cleanser (F68 surfactant), (2) a moisturizer (glycerol, also known as glycerin), and (3) an absorbing agent (superabsorbent starch copolymer), all in the polyurethane matrix. Both F68 and glycerol are soluble in wound fluid or skin moisture. The superabsorbent starch copolymer contained in PolyMem draws wound fluid, which is known to contain natural growth factors and nutrients, to the wound site.

INDICATIONS For the management of:

- Pressure ulcers (Stages I - IV)
- Venous stasis ulcers
- Acute wounds
- Leg ulcers
- Donor and graft sites
- Skin tears
- Diabetic ulcers
- Dermatologic disorders
- 1st and 2nd degree burns
- Surgical wounds

CONTRAINDICATIONS

PolyMem dressings are not compatible with hypochlorite solutions (bleach).

DIRECTIONS FOR USE

APPLICATION

1. **FIRST PolyMem DRESSING APPLICATION:** Prepare the wound according to protocol or as directed by a physician or other ordering clinician.
2. Select a PolyMem dressing so that the membrane pad should be approximately 1/4 – 2" (0.6 – 5 cm) larger than the wound.
3. Apply the dressing film side out and/or printed side out. Medical fixation tape may also be applied to secure the dressing to the surrounding skin.
4. **DRAMATIC** increase in wound fluid may be observed during the first few days. This is not uncommon, and indicates that the PolyMem dressing is working.
HELPFUL HINT: Outlining the wound size on the outside of the dressing helps in determining when a dressing change is needed
5. Keep the PolyMem dressing DRY and in place when bathing. Change the dressing if it gets wet.
6. Topical treatments are not recommended in conjunction with PolyMem dressings.

DRESSING CHANGE

1. Change PolyMem dressing before the wound fluid, visible through the back of the dressing, reaches the wound margin. Change the dressing immediately if the wound fluid reaches the edge of the dressing. For mildly exuding wounds in otherwise healthy patients, dressing can be left on for a recommended maximum of seven days. As with other dressings, more frequent changes may be indicated if the patient has a compromised immune system, diabetes, or infection at the wound site.
2. Gently remove the PolyMem dressing from the wound site. PolyMem will not adhere to the wound bed, usually assuring pain-free dressing changes.
3. **Do not disturb the wound bed. Do not clean the wound bed** or flush with saline or water unless the wound is infected or contaminated. PolyMem dressings contain a wound cleanser and leave no residue. Additional cleaning of the wound may injure regenerating tissue and delay wound healing. The intact skin around the wound may be cleansed while leaving the wound bed undisturbed.
4. Apply a new PolyMem dressing.

The PolyMem Family of Products Relieves Pain.

Ferris PolyMem is suitable for use when visible signs of infection are present if proper medical treatment that addresses the cause of the infection has been implemented.

STORAGE: Keep dry and store at room temperature and humidity.

DISPOSAL: Dispose of used Ferris dressings as typical biohazardous plastic material.

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