
PolyMem®

The Pink Dressing®

BRIEF HOW TO USE:

APPLICATION

- 1. INITIAL DRESSING APPLICATION:** Prepare the wound according to protocol or as directed by a physician or other ordering clinician.
- Select a dressing so that the membrane is approximately 1/4" – 2" (0.6 – 5 cm) larger than the wound. **HELPFUL HINT:** For dry wounds, moisten dressing slightly with sterile water or saline prior to application.
- Apply the dressing film side out and/or printed side out.
- 4. DRAMATIC** increase in wound fluid may be observed during the first few days. This is not uncommon, and indicates that the dressing is working. **HELPFUL HINT:** Outlining the wound size on the outside of the dressing helps in determining when a dressing change is needed.
- Keep the dressing **DRY** and in place when bathing. Change the dressing if it gets wet.
- Topical treatments are not recommended in conjunction with PolyMem family of dressings.

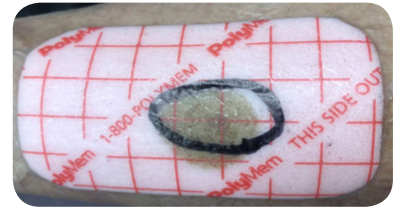DRESSING CHANGE

- Change dressings before the wound fluid, visible through the dressing, reaches the wound margin. Change the dressing immediately if the wound fluid reaches the edge of the membrane pad. For mildly exuding wounds in otherwise healthy patients, dressing can be left on for a recommended maximum of seven days. As with other dressings, more frequent changes may be indicated if the patient has a compromised immune system, diabetes, or infection at the wound site.
- If using tape, gently peel the adhesive toward the center of the dressing.
- Remove the dressing from the wound site. The PolyMem pad will generally not adhere to the wound bed, usually assuring pain-free dressing changes.
- In most cases, when using PolyMem there is no need to disturb or cleanse the wound bed unless the wound is infected or contaminated. PolyMem is designed to continuously clean the wound and does not leave residue that needs to be removed. Excessive cleaning may injure regenerating tissue and delay wound healing. Clean intact skin around the wound while leaving the wound bed undisturbed.
- Apply a new dressing.

Instructions for Use found in each product box should be referenced for specific information and use.

Ferris Mfg. Corp. 5133 Northeast Parkway, Fort Worth, TX 76106-1822 USA | 800-POLYMEM (765-9636) | www.polymem.com

Unless otherwise indicated, all trademarks are owned by or licensed to Ferris.
© 2012-2018 Ferris Mfg. Corp. MKL-009 R13 1118

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